ORIGIO® Gradient™ 100

Product No.:

8400

Languages:

Customer Service:

E-mail: sales@coopersurgical.com

ORIGIO a/s
Knardrupvej 2, 2760 Måløv, Denmark
www.fertility.coopersurgical.com
Tel.: +45 46 79 02 00

origio

a CooperSurgical Company

Ingredientia

8400 ORIGIO® Gradient™ 100
Aqua; CaCl₂; Glucosum; Taurinum; Acidum edeticum; HEPES; KCl; MgSO₄; NaCl; NaHCO₃; NaH₂PO₄; Natrii citras; Pyruvas; Silanii silica colloidalis

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- Balauria A, et al., Processes involved in assisted reproduction technologies significantly increase sperm DNA fragmentation and phosphatidylserine translocation. Andrologia. 2014 Mar; 46(2):86-97.

en - english

For the efficient separation of motile sperm from the ejaculate by the density gradient method.

This product is for ART treatment, whether the cause of infertility is male or female. The product should only be used by professionals trained in ART treatment.

Quality control testing
Sterility tested (Ph.Eur., USP)
Osmolality tested (Ph.Eur., USP)
pH tested (Ph.Eur., USP)
Endotoxin tested ≤ 0,8 EU/ml (Ph.Eur., USP)
Sperm survival tested
Note: The results of each batch are stated on a Certificate of Analysis, which is available at www.fertility.coopersurgical.com.

Buffer system
ORIGIO® Gradient™ is a fully HEPES-buffered medium and does not require pre-incubation before use.

Storage instructions and stability
The products are aseptically processed and supplied sterile.
Store in original container at 2-8°C, protected from light.
Discard excess (unused) media following warming.
The product is to be used within 28 days after opening.
When stored as directed by the manufacturer the product is stable until the expiry date shown on the label.

Precautions and warnings
Do not use the product if:
1. Product packaging appears damaged or if the seal is broken.
2. Expiry date has been exceeded.
3. The product becomes discoloured, cloudy, turbid, or shows any evidence of microbial contamination.

Note: Please be aware that precipitation (colloidal aggregates) may occur. Decanting the medium is recommended when precipitation is present.

Note: Please note the need for traceability of this product. In addition, national legal requirements for this field may exist in your country.

Note: Only to be used in combination with other devices intended for the particular purpose.

Note: Dispose of the device in accordance with local regulations for disposal of medical devices.

Instructions for use

- For custom gradient preparation, use the chart below to prepare a chosen gradient using ORIGIO® Gradient™ 100 and ORIGIO® Sperm Wash.

ORIGIO® Gradient™ 100	ORIGIO® Sperm Wash	End result
8 ml	2 ml	= 10 ml lower phase 80% solution
4 ml	6 ml	= 10 ml upper phase 40% solution

- Bring all components to room temperature (20-25°C) before use.

- For each 1-2 ml of semen sample used, prepare a separate gradient using 1-2 ml of upper phase (40% ORIGIO® Gradient™) carefully underlain with 1-2 ml of lower phase (80% ORIGIO® Gradient™.) *Optimal results may be achieved if gradients are prepared shortly before use.*

- Carefully dispense up to 2 ml of liquefied semen sample on top of the prepared gradient. *Care may be taken not to overload the gradient, as this may result in a poor sperm yield.*

- The gradient is centrifuged at 300-400 g for 15-20 minutes. *Centrifugation speed and duration may be modified to improve the quality and/or yield of sperm collected, depending on the initial semen characteristics.*

- Remove the supernatant from the pellet and transfer the pellet with a new sterile tip into a clean conical centrifugal test tube containing 3-5 ml of ORIGIO® Sperm Wash.

- Centrifuge at 200-300 g for 5-10 minutes. Aspirate and remove most of the supernatant. Repeat this washing procedure.

- Finally, re-suspend the pellet in an appropriate medium according to further procedure; determine the motility and concentration of spermatozoa.

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² Balasuria A, et al., Processes involved in assisted reproduction technologies significantly increase sperm DNA fragmentation and phosphatidylserine translocation. Andrologia. 2014 Mar; 46(2):86-97.

¹ Jackson R.E, et al., Effects of semen storage and separation techniques on sperm DNA fragmentation. Fertil Steril. 2010;94(7):2626–2630.

² Balasuria A, et al., Processes involved in assisted reproduction technologies significantly increase sperm DNA fragmentation and phosphatidylserine translocation. Andrologia. 2014 Mar; 46(2):86-97.

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