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For retrieval, holding and washing of oocytes.

This product is for ART treatment, whether the cause of infertility is male or female. The product should only be used by professionals trained in ART treatment.

Products
1584 SynVitr[®] Flush
1576 SynVitr[®] Flush with 10 IU/ml heparin

Contains
Heparin 10 IU/ml (product no. 1576 only)
Recombinant human insulin

Quality control testing
Sterility tested (Ph.Eur., USP)
Osmolality tested (Ph.Eur., USP)
pH tested (Ph.Eur., USP)
Endotoxin tested ≤ 0,1 EU/ml (Ph.Eur., USP)
Mouse Embryo Assay (MEA) tested
Note: The results of each batch are stated on a Certificate of Analysis, which is available at www.origio.com.

Storage instructions and stability
The products are aseptically processed and supplied sterile.
Store in original container at 2-8°C, protected from light.
Do not freeze.
Discard excess (unused) media following warming.
The product is provided in vials intended for single use.
When stored as directed by the manufacturer the product is stable until the expiry date shown on the label.

Precautions and warnings
Do not use the product if:
1. Product packaging appears damaged or if the seal is broken.
2. Expiry date has been exceeded.
3. The product becomes discoloured, cloudy, turbid, or shows any evidence of microbial contamination.

Caution: The product is for single use and is not to be reused due to risk of contamination.

Caution: Oocytes are extremely sensitive to pH changes and transient cooling in vitro. Modest fluctuations in temperature can cause disruption of the meiotic spindle with possible chromosome dispersal.

Note: The cap of the bottle should be kept tightly closed during the pre-warming.

Note: Please note the need for traceability of this product. In addition, national legal requirements for this field may exist in your country.

Note: Only to be used in combination with other devices intended for the particular purpose.

Note: Dispose of the device in accordance with local regulations for disposal of medical devices.

Instructions for use
1. Pre-warm SynVitr[®] Flush to 37°C.
2. Keep SynVitr[®] Flush at 37°C in a heating block throughout the entire process of oocyte recovery and transportation to the embryology laboratory.
3. Syringes used for flushing the follicle are filled with the SynVitr[®] Flush. Aspirate the follicular fluid and flush the follicle with a volume of medium equivalent to the amount of follicular fluid collected. Take care not to expand the follicle beyond its original size.
4. Wash oocytes with SynVitr[®] Flush or pre-equilibrated culture medium, taking care to remove any blood clots and granulosa cells.
5. Transfer oocytes to pre-equilibrated culture medium and store at 37°C in a 5-6% CO₂ environment.

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