	<p>EC REP CooperSurgical Distribution B.V. Gallesweg 35, 5522 PR Venlo, The Netherlands LifeGlobal Group, LLC, 393 Soundview Rd, Guilford, CT 06437 USA T: 1800-648-1151 Int.: +45 46790200 sales@coopersurgical.com Orders@origio.us.com</p>
LGGH – V15	

EN	CE <i>2797</i>	Instructions for the Use of  global® w/ HEPES (Catalogue Numbers: LGGH-050, LGGH-100, LGGH-250, LGGH-500)
-----------	--------------------------	--

PRECAUTIONS AND WARNINGS

- Caution:** Federal Law (USA) restricts this device to sale by or on the order of a physician (or properly licensed practitioner).
- Caution:** The user should read and understand the Instructions for Use, Precautions and Warnings, and be trained in the correct procedure before using global® w/ HEPES for embryo washing and handling.
- Not to be used for injection.
- Do not resterilize.
- Do not use the product if:
 - the product packaging appears damaged or if the seal is broken
 - the expiry date has been exceeded
 - the product becomes discolored, cloudy, or shows evidence of particulate matter
- global® w/ HEPES contains only a low concentration of sodium bicarbonate and should not be gassed, or used for culture under CO₂.
- global® w/ HEPES contains the antibiotic gentamicin sulfate. Appropriate precautions should be taken to ensure that the patient is not sensitized to this antibiotic.
- To avoid problems with contamination, practice aseptic techniques.
- Discard unused medium within 7 days of opening. Do not use after expiry date.

GENERAL INFORMATION

Indications for Use

Embryo washing and handling.

Storage and Shelf Life

Store at 2-8°C and protected from light. Ten (10) weeks from the date of manufacture.

Composition

A HEPES-buffered medium is required when embryos are handled outside of a CO₂ incubator.

Sodium Chloride, Potassium Chloride, Calcium Chloride, Potassium Phosphate, Magnesium Sulfate, Sodium Bicarbonate, Glucose, Lactate Na Salt, Sodium Pyruvate, Glycine, L-Alanine, L-Arginine HCl, L-Asparagine, L-Aspartic Acid, L-Cystine, L-Glutamic Acid, Glycyl-Glutamine, L-Histidine, L-Isoleucine, L-Leucine, L-Lysine HCl, L-Methionine, L-Phenylalanine, L-Proline, L-Serine, L-Threonine, L-Tryptophan, L-Tyrosine, L-Valine, EDTA, Phenol Red, HEPES, Gentamicin Sulfate* (10 µg/ml)

*non therapeutic-grade source material

Assay (performed for each batch)	Specification
Physicochemical Tests	
pH	7.2-7.4
Osmolality	260-270 mOsm
Biological Tests	
Endotoxin (LAL)	≤ 0.5 EU/ml
Sterility Test (bacterial and fungal screen, SAL 10 ⁻³)	PASS
Biological Assays	
1-cell Mouse Embryo Assay (% expanded blastocysts at 96 h of culture after 1 h exposure)	≥ 80%

INSTRUCTIONS FOR USE

The procedures described below have been found to be effective for the preparation of global® w/ HEPES for embryo washing and handling. Every laboratory must define and optimize its own procedures.

- Open the bottle and add a sufficient volume of Human Serum Albumin (HSA) or LifeGlobal® Protein Supplement to produce a desired final concentration of HSA or LifeGlobal® Protein Supplement in the medium. Cap and gently agitate the bottle to completely mix the HSA or LifeGlobal® Protein Supplement into the medium.
- Aliquot the volume of supplemented global® w/ HEPES needed for one patient or one day into appropriate embryo-safe culture tubes. Cap the tubes tightly and store at 2-8°C, protected from light.
- To wash oil, place 50 ml of global® w/ HEPES (without HSA or LifeGlobal® Protein) into a tissue culture flask and overlay with 250 ml of oil. Agitate flask. Store at 2-8°C.
- The night before intended use aliquot a sufficient volume of washed oil into embryo-safe culture tubes. Cap the tubes and place them in a heating block at 37° C, overnight.
- The night before or the morning of intended use, place a sufficient number of tubes of supplemented global® w/ HEPES into a heating block at 37°C.
- Prepare a sufficient number of plates containing micro-drops of supplemented global® w/ HEPES under washed oil.
- Cover and seal the dishes and place into the incubator for final temperature equilibration for 1-2 hours.

SYMBOLS

STERILE A	RX Only	REF	LOT			
Sterile Using Aseptic Processing Techniques	By Prescription Only	Catalogue Number	Batch Code	Consult Instructions For Use	Manufacturer	Keep Away From Sunlight
	EC REP				CE <i>2797</i>	
Temperature Limitation	Authorized Representative in the European Community	Use By	GSI DataMatrix Barcode	Do Not Resterilize	European Conformance (notified body)	Do Not Use if Package is Damaged

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