

## ZyMōt® MULTI SPERM SEPARATION DEVICE

---

An advanced method for sperm separation




# Male factor infertility


Male factor infertility contributes to approximately 50% of all infertility cases, highlighting the **importance of sperm quality and health**.<sup>1,2</sup>

Sperm are designed to deliver DNA and are unique cells with distinctive morphology (shape). The number, motility (ability to swim), morphology and integrity of sperm DNA can impact your fertility treatment and success.


## Examples of sperm disorders

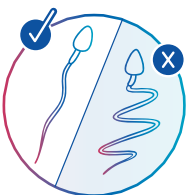


**Low sperm motility**  
Reduced proportion of sperm able to swim or sperm with poor quality of movement

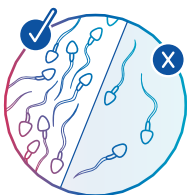

**Low sperm count**  
Reduced numbers of sperm

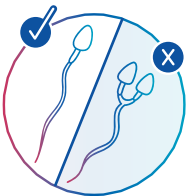

**Abnormal sperm morphology**  
Misshapen sperm with defects to head and or tail

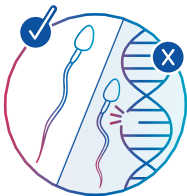

**Sperm DNA fragmentation**  
Disturbance to the genetic material of the sperm

**ZyMöt represented a fundamental change in thinking about how to effectively separate high-quality sperm for ART procedures.**<sup>3-5</sup>

## What is sperm separation?

During fertility treatments, an embryologist visually evaluates the semen sample's appearance and sperm motility and selects the most motile sperm, to help ensure the best chance of success. However, visual inspection alone cannot always reveal immature or damaged sperm and if unsuitable sperm are chosen, they might affect embryo quality and may increase the risk of pregnancy loss.<sup>6</sup>

### What is the ZyMöt Multi Sperm Separation Device?



The ZyMöt® device is a first-of-its-kind patented innovation<sup>7</sup> and helps retrieve a high-quality motile sperm sample with considerably lower levels of DNA damage.<sup>3-5</sup>

It is an innovative and effective way to separate motile sperm for fertility treatments that helps to increase the chances of successful implantation and pregnancy.<sup>8</sup> The ZyMöt Multi Sperm Separation Device uses only the most motile sperm for ART procedures, whether ICSI, IVF or IUI.




## How does ZyMöt® work?


ZyMöt creates a competitive environment for sperm separation.<sup>3</sup> It has two chambers separated by a microporous membrane that filter high-quality sperm, isolating them in the upper chamber.<sup>3-5</sup> The most motile sperm can swim through the pores of the membrane and can then be harvested from the upper chamber by the embryologist. Immature (not fully formed) sperm, damaged sperm or those sperm with low motility (not good at swimming) do not reach the end destination.

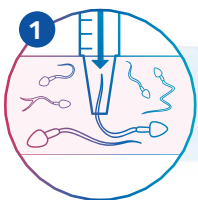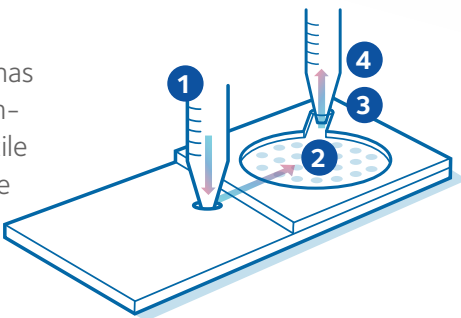


The sperm sample is injected into the lower chamber via the inlet port

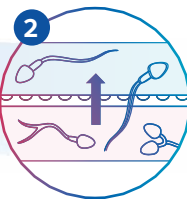

The most motile and viable sperm can swim through the holes in the microporous membrane and into the upper chamber

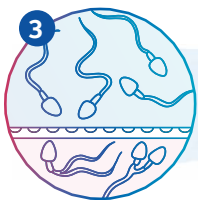

The most motile and viable sperm are isolated in the upper chamber

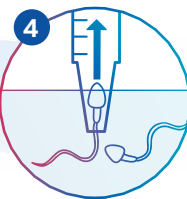

Your embryologist collects the sperm for your fertility treatment

## How can the proven success of the ZyMöt® device help me?<sup>9,10</sup>

One motile sperm and one egg are all that is needed to begin the process of creating an embryo. The quality of the sperm used matters, and when processing with the ZyMöt device, it enhances the recovery of sperm with good motility, progression and morphology, along with providing a significant reduction of DNA damage.<sup>9,10</sup>

For men with high sperm DNA fragmentation, ZyMöt-processed samples have been shown to select high-quality, motile sperm that helps enhance fertilization, embryo development, and pregnancy rates.<sup>8</sup>

**Using the ZyMöt device has been shown to increase the number of euploid embryos and can potentially lead to pregnancies for couples with a history of previous ART failure.**<sup>8,11</sup>



# Talk to your clinician

ZyMöt can help make it easier for your fertility team to isolate high-quality, motile sperm<sup>3-5</sup> to help improve your chances of a successful treatment.<sup>8-10</sup>



ART, assisted reproductive technology; ICSI, intracytoplasmic sperm injection ; IVF, *in vitro* fertilization; IUI, intrauterine insemination.

**Contact your fertility clinician to find out if ZyMöt is a good option for you, or visit our website to learn more:**



## References

1. Calvert, J.K. *et al.*, 2022. The male infertility evaluation still matters in the era of high efficacy assisted reproductive technology. *Fertility and Sterility*, 118(1), pp.34-46
2. Agarwal, A. *et al.*, 2015. A unique view on male infertility around the globe. *Reproductive Biology and Endocrinology*, 13, pp.1-9
3. Asghar, W. *et al.*, 2014. Selection of functional human sperm with higher DNA integrity and fewer reactive oxygen species. *Advanced Healthcare Materials*, 3(10), pp.1671-1679
4. Broussard, A. *et al.*, 2019. Sperm DNA fragmentation (SDF) was most effectively improved by a sperm separation device compared to different gradient and swimup methods. *Fertility and Sterility*, 111(4), p.e15
5. Bastuba, M. *et al.*, 2020. Microfluidic sperm separation device dramatically lowers DFI. *Fertility and Sterility*, 113(4), p.e44
6. Tan, J. *et al.*, 2019. Association between sperm DNA fragmentation and idiopathic recurrent pregnancy loss: a systematic review and meta-analysis. *Reproductive Biomed Online*, 38(6), 951
7. ZyMöt™ Multi Sperm Separation Device Instructions for Use (IFU)
8. Kocur, O.M. *et al.*, 2023. Can a sperm selection technique improve embryo ploidy? *Andrology*, 11(8), pp.1605-1612
9. Gode, F. *et al.*, 2019. Comparison of microfluid sperm sorting chip and density gradient methods for use in intrauterine insemination cycles. *Fertility and Sterility*, 112(5), pp.842-848
10. Parrella, A. *et al.*, 2018. Microfluidic selection of spermatozoa retains chromatin integrity and yields higher pregnancy rates. *Fertility and Sterility*, 110(4), p.e343
11. Anderson, A. 2020. Euploidy rates and pregnancy outcomes using the ZyMöt™ device for sperm separation. *ASRM 2020 Virtual Scientific Congress & Expo ART-LAB Inter-professional Collaboration*