
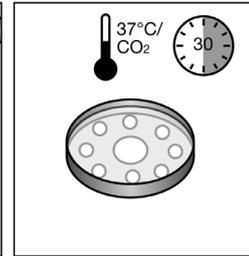
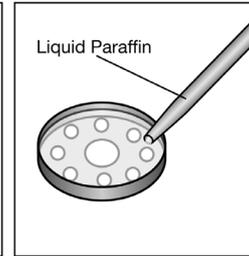
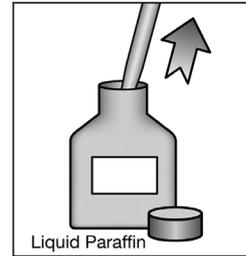


Liquid Paraffin

Product No.:

1010

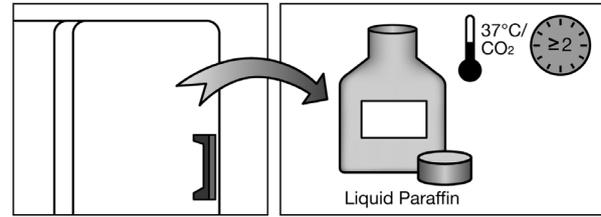

Customer Service:

E-mail:

customerservice.medicult@origio.com

Tel: +45 46 79 02 02 • Fax: +45 46 79 03 02

ORIGIO a/s

Knardrupvej 2, DK-2760 Måløv, Denmark

www.origio.com

Tel: +45 46 79 02 00 • Fax: +45 46 79 03 00

Liquid paraffin

Intended use

Liquid Paraffin is used as an oil overlay for culture media during IVF and ICSI procedures.

Composition

Liquid paraffin Ph.Eur., pre-washed with a solution containing HSA.

Quality control testing

Sterility tested (Ph.Eur., USP)

Endotoxin tested ≤ 0.1 EU/mL (Ph.Eur., USP)

1 cell Mouse Embryo Assay (MEA) $\geq 80\%$

Blastocysts by 96h

Sperm Survival tested

Note: The results of each batch are stated on a Certificate of Analysis, which is available on www.origio.com.

Storage instructions and stability

Store in original container at 2-8°C, protected from light.

Do not freeze.

Discard excess (unused) media following warming.

The product is to be used within 28 days after opening.

When stored as directed by the manufacturer the product is stable until the expiry date shown on the vial label.

Precautions and warnings

Do not use the product if:

1. Product packaging appears damaged or if the seal is broken.
2. Expiry date has been exceeded.

Caution: During the manufacturing process the product is washed with medium containing small amounts of human serum albumin (HSA), which has been obtained from a U.S. licensed source.

All blood products should be treated as potentially infectious. Source material used to manufacture this product were tested and found non-reactive for HbsAg and negative for Anti-HIV-1/-2, HIV-1, HBV, and HCV.

Furthermore source material have been tested for parvovirus B19 and found to be non-elevated. No known test methods can offer assurances that products derived from human blood will not transmit infectious agents.

Caution: US federal law restricts this device to sale by or on the order of a physician (Rx only).

Instructions for use

1. Place the Liquid Paraffin in an incubator with a 5-6% CO₂ environment at 37°C for at least 2 hours prior to use.
2. Pipette the intended volume of medium to be used for the microdrop on the bottom of a petri dish.
3. Gently pipette the equilibrated Liquid Paraffin into the petri dish until the microdrops are sufficiently covered with oil.